


MedGyn Silicone Pessaries

Pessaries are an effective tool for the non-surgical treatment of a variety of gynecologic problems. Commonly used in management of pelvic support defects, pessaries are also used to treat stress urinary incontinence. MedGyn offers an extensive line of quality silicone pessaries and an easy to use ring pessary fitting set



The Pessary Fitting Set is an ideal diagnostic tool to help the clinician determine the proper style and size of pessary for each patient. Stored in an attractive case, the set includes six popular Ring pessaries and an embossed cross-reference chart to use in selecting the appropriate pessary. One set may be used for multiple patients as the latex-free rings are autoclaveable. Methods for cleaning are included with every pessary set.

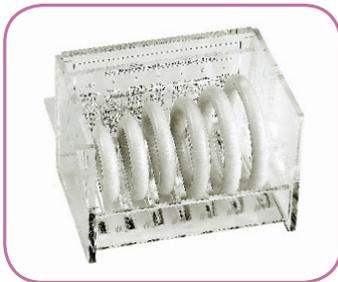


The Cube Pessary, available with or without drains, may be used to relieve the symptoms of a cystocele or a rectocele as well as a second or third-degree uterine prolapse.



The Cup Pessary, available with a support membrane, is used for a first or a mild second-degree uterine prolapse.



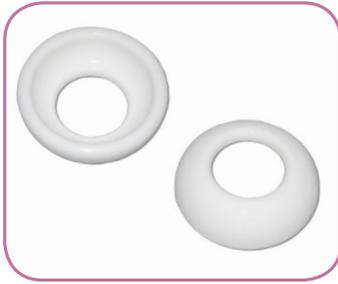

The Cerclage Pessary is designed to support an incompetent cervix and shift the cervix toward sacrum. This pessary may be used to support women with a history of premature labor or multiple pregnancies.



The Donut Pessary may be used to relieve the stress of a second or third-degree uterine prolapse, a cystocele or rectocele. This pessary, often used by post-menopausal women, is easily compressed for insertion and removal.



The Dish pessary, available with a supporting membrane, is used to relieve the stress of a second or third-degree uterine prolapse, a cystocele or rectocele. This pessary may also relieve stress urinary incontinence.



The Gellhorn Pessary with drain feature is used to relieve the symptoms of a cystocele or a rectocele as well as a second or third-degree uterine prolapse. The pessary stem with knob is easily folded for insertion and removal. The Short-Stem may provide a better fit option for patients.



The Oval Pessary, available with a supporting membrane, is commonly used for a mild first-degree uterine prolapse or a cystocele. This pessary's shape inhibits its rotation in women with a history of vaginal surgery.



The Ring Pessary, available with or without a supporting membrane, is primarily used for mild first-degree uterine prolapse or a cystocele.



The Knobbed Ring pessary, available with or without a support membrane, has the same indications as the Ring pessary and may also relieve stress urinary incontinence.



The Shatz Pessary with drain may relieve the symptoms of a cystocele as well as a second or third-degree uterine prolapse. Its convex shape ensures a snug fit while the drainage holes permit easy removal.



The Marland Pessary is available with or without support membrane, is used to relieve the symptoms of a second or third-degree uterine prolapse, a cystocele or rectocele.



The Hodge Pessary is available with or without support membrane, commonly used for a first or mild second-degree uterine prolapse and may also relieve a cystocele. A Hodge Pessary with support is also effective in the prevention of exercise incontinence.



The Gehring Pessary is available with support membrane is primarily used for a significant cystocele or rectocele and may also provide relief for second or third-degree uterine prolapse.

| Pessary | 1st - 2nd Degree | 2nd - 3rd Degree | Sexually Active | Vaginal Wall | Stress Urinary Incontinence | Cystocele | Rectocele | Incompetent Cervix |
|---------------------------|------------------|------------------|-----------------|--------------|-----------------------------|-----------|-----------|--------------------|
| Cube | | X | X | X | X | X | Mild X | |
| Cup w/ Support | X | | | | | | | |
| Cerclage | | X | | | | X | | X |
| Donut | | X | | | | Mild X | | |
| Dish w/ Support | X | | | | X | Mild X | | |
| Gellhorn | | X | | | | Mild X | | |
| Oval w/ Support | X | | | | | X | | |
| Ring | X | | X | | | | | |
| Ring w/ Support | X | | X | | | Mild X | | |
| Ring w/ Knobbed | X | | X | | X | Mild X | | |
| Ring w/ Knobbed & Support | X | | X | | X | Mild X | | |
| Shatz w/ Drain | X | | | | | Mild X | | |
| Marland w/ Support | | X | | | | X | X | |
| Marland w/o Support | | X | | | | X | X | |
| Hodge w/ Support | X | | | | | X | | |
| Hodge w/o Support | X | | | | | X | | |
| Gehrung w/ Support | | X | | | | X | X | |



Pessary Sets

- 051000 Pessary Fitting Set of 6
- 051005 Ring w/ Support Pessary Fitting Set of 6
- 051006 Shaatz Pessary Fitting Set of 6

Cube

- 050066 Cube w/ drain #0
- 050067 Cube w/ drain #1
- 050068 Cube w/ drain #2
- 050069 Cube w/ drain #3
- 050070 Cube w/ drain #4
- 050071 Cube w/ drain #5
- 050072 Cube w/ drain #6
- 050073 Cube w/ drain #7
- 050074 Cube w/ drain #8
- 050075 Cube w/ drain #9
- 050076 Cube w/ drain #10
- 050055 Cube w/o drain #0
- 050056 Cube w/o drain #1
- 050057 Cube w/o drain #2
- 050058 Cube w/o drain #3
- 050059 Cube w/o drain #4
- 050060 Cube w/o drain #5
- 050061 Cube w/o drain #6
- 050062 Cube w/o drain #7
- 050063 Cube w/o drain #8
- 050064 Cube w/o drain #9
- 050065 Cube w/o drain #10

Cup

- 050171 Cup w/ support #0
- 050172 Cup w/ support #1
- 050173 Cup w/ support #2
- 050174 Cup w/ support #3
- 050175 Cup w/ support #4
- 050176 Cup w/ support #5
- 050177 Cup w/ support #6
- 050178 Cup w/ support #7

- 050163 Cup w/o support #0
- 050164 Cup w/o support #1
- 050165 Cup w/o support #2
- 050166 Cup w/o support #3
- 050167 Cup w/o support #4
- 050168 Cup w/o support #5
- 050169 Cup w/o support #6
- 050170 Cup w/o support #7

Cerclage

- 050210 Cerclage w/ drain #1
- 050211 Cerclage w/ drain #2
- 050212 Cerclage w/ drain #3
- 050213 Cerclage w/ drain #4
- 050214 Cerclage w/ drain #5
- 050215 Cerclage w/ drain #6
- 050191 Cerclage w/o drain #1
- 050192 Cerclage w/o drain #2
- 050193 Cerclage w/o drain #3
- 050194 Cerclage w/o drain #4
- 050195 Cerclage w/o drain #5
- 050196 Cerclage w/o drain #6

Donut

- 050010 Donut #0
- 050011 Donut #1
- 050012 Donut #2
- 050013 Donut #3
- 050014 Donut #4
- 050015 Donut #5
- 050016 Donut #6
- 050199 Donut #7

Dish

- 050137 Dish w/ support #0
- 050138 Dish w/ support #1
- 050139 Dish w/ support #2
- 050140 Dish w/ support #3
- 050141 Dish w/ support #4
- 050142 Dish w/ support #5
- 050143 Dish w/ support #6

- 050144 Dish w/ support #7
- 050129 Dish w/o support #0
- 050130 Dish w/o support #1
- 050131 Dish w/o support #2
- 050132 Dish w/o support #3
- 050133 Dish w/o support #4
- 050134 Dish w/o support #5
- 050135 Dish w/o support #6
- 050136 Dish w/o support #7

Gellhorn

- 050155 Gellhorn w/ drain #1
- 050156 Gellhorn w/ drain #2
- 050157 Gellhorn w/ drain #3
- 050158 Gellhorn w/ drain #4
- 050159 Gellhorn w/ drain #5
- 050160 Gellhorn w/ drain #6
- 050161 Gellhorn w/ drain #7

Oval

- 050111 Oval w/ support #1
- 050112 Oval w/ support #2
- 050113 Oval w/ support #3
- 050114 Oval w/ support #4
- 050115 Oval w/ support #5
- 050116 Oval w/ support #6
- 050117 Oval w/ support #7
- 050118 Oval w/ support #8
- 050119 Oval w/ support #9
- 050101 Oval w/o support #1
- 050102 Oval w/o support #2
- 050103 Oval w/o support #3
- 050104 Oval w/o support #4
- 050105 Oval w/o support #5
- 050106 Oval w/o support #6
- 050107 Oval w/o support #7
- 050108 Oval w/o support #8
- 050109 Oval w/o support #9

Ring

050026 Ring w/ support #1
050027 Ring w/ support #2
050028 Ring w/ support #3
050029 Ring w/ support #4
050030 Ring w/ support #5
050031 Ring w/ support #6
050032 Ring w/ support #7
050033 Ring w/ support #8
050034 Ring w/ support #9
050017 Ring w/o support #1
050018 Ring w/o support #2
050019 Ring w/o support #3
050020 Ring w/o support #4
050021 Ring w/o support #5
050022 Ring w/o support #6
050023 Ring w/o support #7
050024 Ring w/o support #8
050025 Ring w/o support #9

Ring with Knob

050027K Ring w/ knob - w/ support #2
050028K Ring w/ knob - w/ support #3
050029K Ring w/ knob - w/ support #4
050030K Ring w/ knob - w/ support #5
050031K Ring w/ knob - w/ support #6
050032K Ring w/ knob - w/ support #7
050018K Ring w/ knob - w/o support #2
050019K Ring w/ knob - w/o support #3
050020K Ring w/ knob - w/o support #4
050021K Ring w/ knob - w/o support #5

050022K Ring w/ knob - w/o support #6

050023K Ring w/ knob - w/o support #7

Shaatz

050120 Shaatz w/ drain #0
050121 Shaatz w/ drain #1
050122 Shaatz w/ drain #2
050123 Shaatz w/ drain #3
050124 Shaatz w/ drain #4
050125 Shaatz w/ drain #5
050126 Shaatz w/ drain #6
050127 Shaatz w/ drain #7
050128 Shaatz w/ drain #8

Marland Pessary

050084 Marland w/ support #2
050085 Marland w/ support #3
050086 Marland w/ support #4
050087 Marland w/ support #5
050088 Marland w/ support #6
050089 Marland w/ support #7
050090 Marland w/ support #8
050077 Marland w/o support #2
050078 Marland w/o support #3
050079 Marland w/o support #4
050080 Marland w/o support #5
050081 Marland w/o support #6
050082 Marland w/o support #7
050083 Marland w/o support #8

Hodge Pessary

050045 Hodge w/ support #0
050046 Hodge w/ support #1

050047 Hodge w/ support #2

050048 Hodge w/ support #3

050049 Hodge w/ support #4

050050 Hodge w/ support #5

050051 Hodge w/ support #6

050052 Hodge w/ support #7

050053 Hodge w/ support #8

050054 Hodge w/ support #9

050035 Hodge w/o support #0

050036 Hodge w/o support #1

050037 Hodge w/o support #2

050038 Hodge w/o support #3

050039 Hodge w/o support #4

050040 Hodge w/o support #5

050041 Hodge w/o support #6

050042 Hodge w/o support #7

050043 Hodge w/o support #8

050044 Hodge w/o support #9

Gehrung Pessary

050145 Gehrung w/ support #0

050146 Gehrung w/ support #1

050147 Gehrung w/ support #2

050148 Gehrung w/ support #3

050149 Gehrung w/ support #4

050150 Gehrung w/ support #5

050151 Gehrung w/ support #6

050152 Gehrung w/ support #7

050153 Gehrung w/ support #8

TRUSTED PRODUCTS FOR PREMIER WOMEN'S HEALTHCARE

MEDGYN PRODUCTS, INC.

100 W. Industrial Rd. Addison, IL 60101 USA

t: +1 630.627.4105 toll-free: 800.451.9667

f: +1 630.627.0127

e: info@medgyn.com

w: medgyn.com

PESSARY1-PF2

