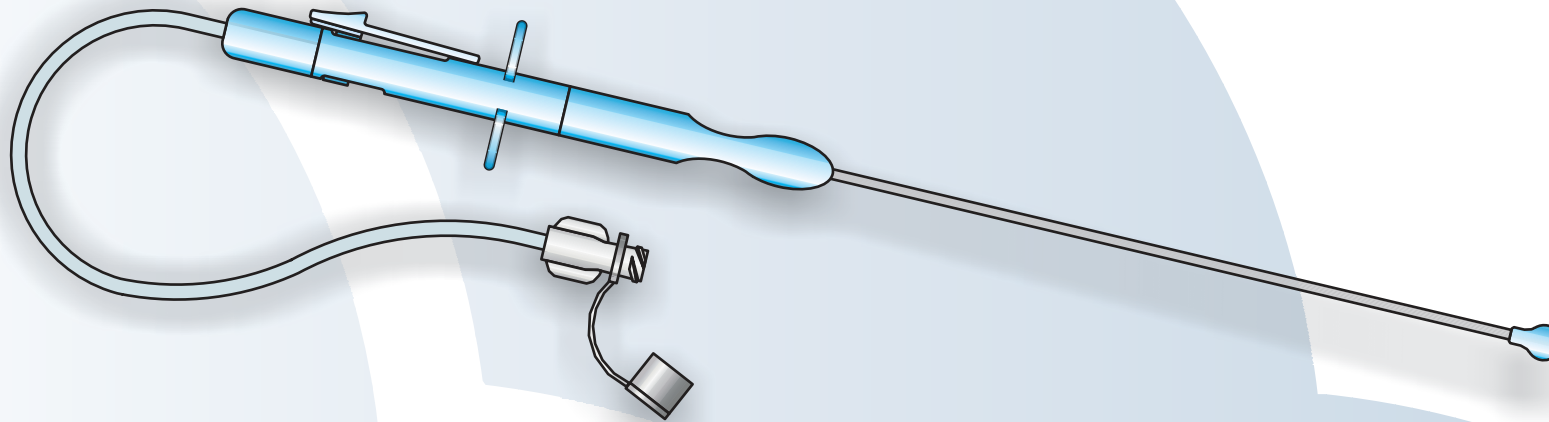
# Vblock®

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Sterile e monouso

Device designed and realized for an easy, safe, reliable and successful execution of the uterine paracervical block.

It consists of:

- a handle at the proximal end of which a slide mechanism is placed to permit a safe and precise release of the needle tip outside a protective sheath; a red marker indicates the right and definitive release of the needle tip (this marker is hidden by the slide mechanism when at rest). The needle tip will return inside the protective sheath pushing the button placed just at the opposite side of the slide mechanism.
- A stainless steel cannula (protective sheath) inside which the needle to perform the procedure is placed. The distal end of this cannula presents a protective plastic ball, so it is possible to ensure a steady and atraumatic contact with the junction of the cervix and the vagina
- 20 Ga, oblique tip stainless steel needle. The release of the needle tip outside the protective plastic ball is 4 mm (and the red marker appears)
- PVC, extension line, 200 mm. length with female Luer lock and cap.
- the total length of the device is 270 mm (the PVC extension is excluded)

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